

- Revolutionary & Healthy Approach
- No petrochemicals
- Long lasting hold 24h*
- Silky smooth hold



The application of a denture adhesive cream implies its absorption through the mouth, into the body. This is why we have decided to replace oils derived from petrochemicals (petrolatum, mineral oil and vaseline), commonly used in the formulation of denture adhesive creams on the market, by a healthier alternative, Olive oil as well as a long lasting hold for up to and over 24 hours*.

A test was conducted with denture wearers by Dermatest (Germany) to assess the effectiveness of OlivaFix. The results were a success and showed that the cream was dermatologically safe, but also had a strong hold, depending on the type of denture.

Due to the high concentration of Olive oil, OlivaFix has a silky consistency and spreads smoothly on the appliance creating a protective film for the gums.

- 0% Silicones
- 0% Colour additives
- 0% Preservatives
- 0% Zinc
- 0% Parabens
- 0% Petrochemicals



Upper denture



Lower denture

24h

12h

6h



For upper dentures, adjust cream quantity to adjust hold time.

* Up to and over 24h for full upper dentures

Instructions for use




1 Before applying the cream clean your denture as usual.



2 Remove the metallic anti-humidity & hygienic seal from the tube.



3 Humidify your dentures and apply the cream as shown in the picture above.

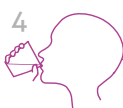

4 Before placing your dentures in the mouth, drink a sip of water to humidify the gums.



5 Place your dentures in the mouth and hold them in place by maintaining pressure for a few seconds.



6 Wait 5 minutes before drinking or eating.

Useful information

It is possible that cream residue may remain on your dentures or your gums after removal. This can be removed by using a cotton bud soaked in warm Olive oil. You can safely adapt the quantity of cream used: lower dentures or flat gums usually require more cream.

Storage: Do not expose the tube to sources of heat or direct sunlight.

Due to the high concentration of Olive oil (up to 35%) it is possible that you notice some Olive oil residue during application. This will in no way affect the performance of the cream nor does it pose any risk to your appliance. In order to minimise these residues, it is recommended to knead the tube before use (between 10 and 30 seconds).

bonyfAG

Heiligkreuz 16
FL-9490 Vaduz
Liechtenstein • EUROPE
www.bonyf.com



Caution: Do not use product if tube nozzle is unsealed. Use the product according to instructions for use. Keep out of the reach of children. Do not use if there is a known sensitivity or allergy to one of the ingredients. If you are taking medication that may affect your gums or oral cavity, consult a dentist before using the product. In the event of gum irritation or soreness, discontinue use and consult your dentist if necessary. Pain and discomfort of the gums can also be caused by ill-fitting or broken dentures therefore it is important to consult your dentist on a regular basis. Store dry and at room temperature.

Ingredients: Cellulose Gum, Olea Europaea (Olive Fruit) Oil, Calcium/Sodium PVM/MA Copolymer, Hydrogenated Soybean Oil, Trihydroxystearin, Silica, Menthol, Lecithin, Citrus Limon Peel Oil, Menthyl Lactate.